

DENOPS

DENTIS No Pain System

Give your patients
a pain-free experience!



**SET
YOUR CLINIC
APART**

DENOPS

DENTIS No Pain System




Plastic Ampule Case With Silicon Control Button

- Control injection speed and suction function
- Check injection volume and suction function with transparent ampule case and window
- Autoclave material available for preventing cross infection

- **Light and slim** design for easy use
- Injection **speed control** (Low / Mid / High)
- **Easy operation** with 'User Mode'
- **Voice** assistant

LCD menu selection and mode change

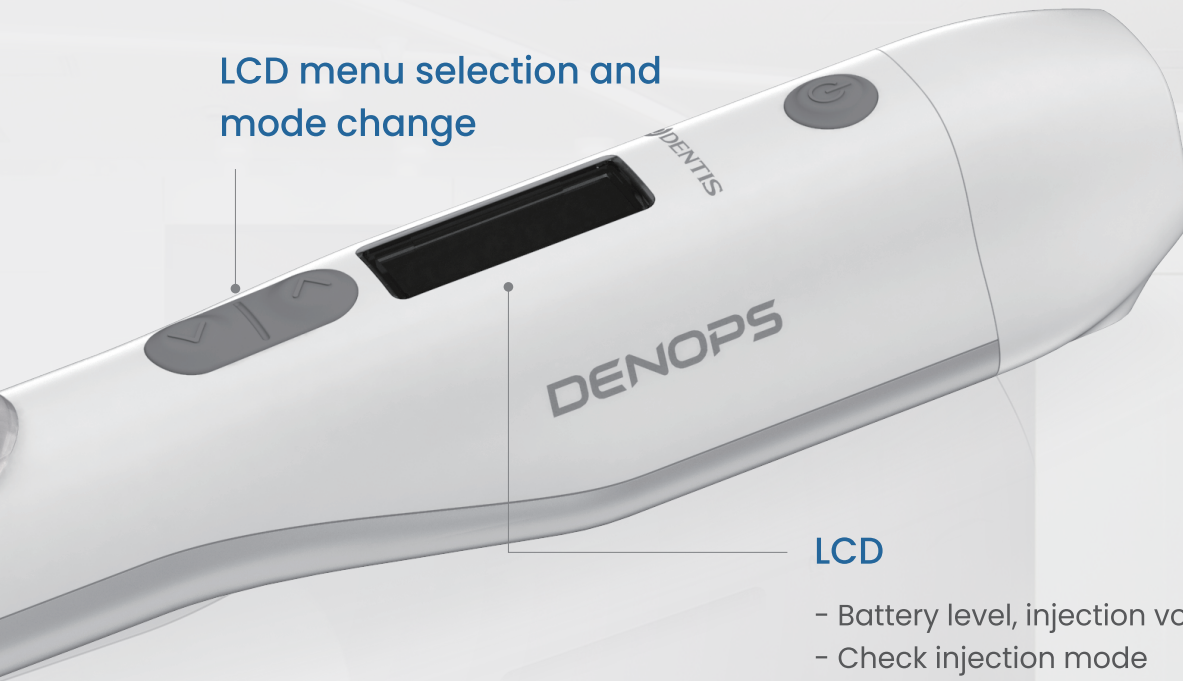

LCD

- Battery level, injection volume and speed display
- Check injection mode



Remaining volume

Volume

Speed

Mode

Wireless charging using a dedicated charging cradle

- Inclined design prevents leakage of anesthetic solution
- Battery level indicator





Ampule Case (3 EA)




Main Body



Charging Cradle



Charging Adapter

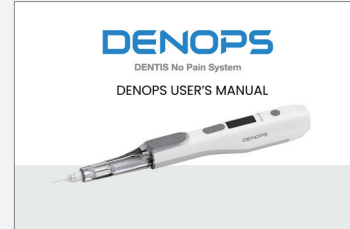

User Manual

Components

Parts	Category	Unit	Specification	Remark	
Main Body (+ Ampule Case)	Weight	g	85		
	Injection Volume	ml	5 Stages (Checking with LCD on Main Body) 1/16 : 0.11 ml, 1/8 : 0.22 ml, 1/4 : 0.45 ml, 1/2 : 0.90 ml, FULL : 1.8 ml		
	Injection Time	s	Manual – 1st stage : 320s, 2nd stage : 160s 3rd stage : 30s Auto – 1st stage : 320s, 2nd stage : 170s 3rd stage : 48s	When injecting 1.8 ml	
	Injection Mode	-	AUTO, MANUAL, USER		
	Battery	V		DC 3.7	Lithium Polymer
		mAh		550	
	Charging Time	h		6	
Size	mm		202.5 X 24.7 X 28		
Charging Cradle	Weight	g	140		
	Size	mm	176 X 41.4 X 75.3		
Charging Adapter	Rated Input	V	AC 100 – 240		
		Hz	50 – 60		
	Rated output	V	DC 5		
		A	2.0		
	Weight	g	104		
Dental Needle (Disposable) <i>*Separate Purchase</i>	Size	mm	27G/16, 25, 30 30G/ 12, 16, 25 31G/ 12		

