


⊕ Fixation Components (For Separate Purchase)

Bone Screw

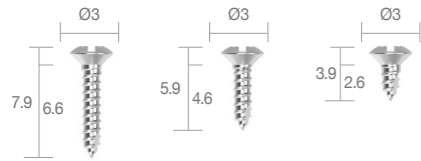


Spec	Bone Screw
Length	Ø3 for common use
7mm	DBS1407
5mm	DBS1405
3mm	DBS1403

Bone Tack




Spec	Bone Tack
Length	Ø2.5 for common use
5mm	DBT2505
3.5mm	DBT2503

Healing Cap

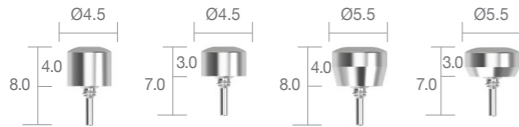



Spec	Healing Cap	
Length	Ø4.5	Ø5.5
4mm	DGHCN4504	DGHCN5504
3mm	DGHCN4503	DGHCN5503

Spacer

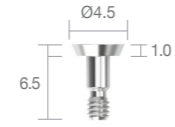

Spec	Spacer
Length	Ø4.5
1mm	DGSN4501

⊕ Easy & Simple GBR

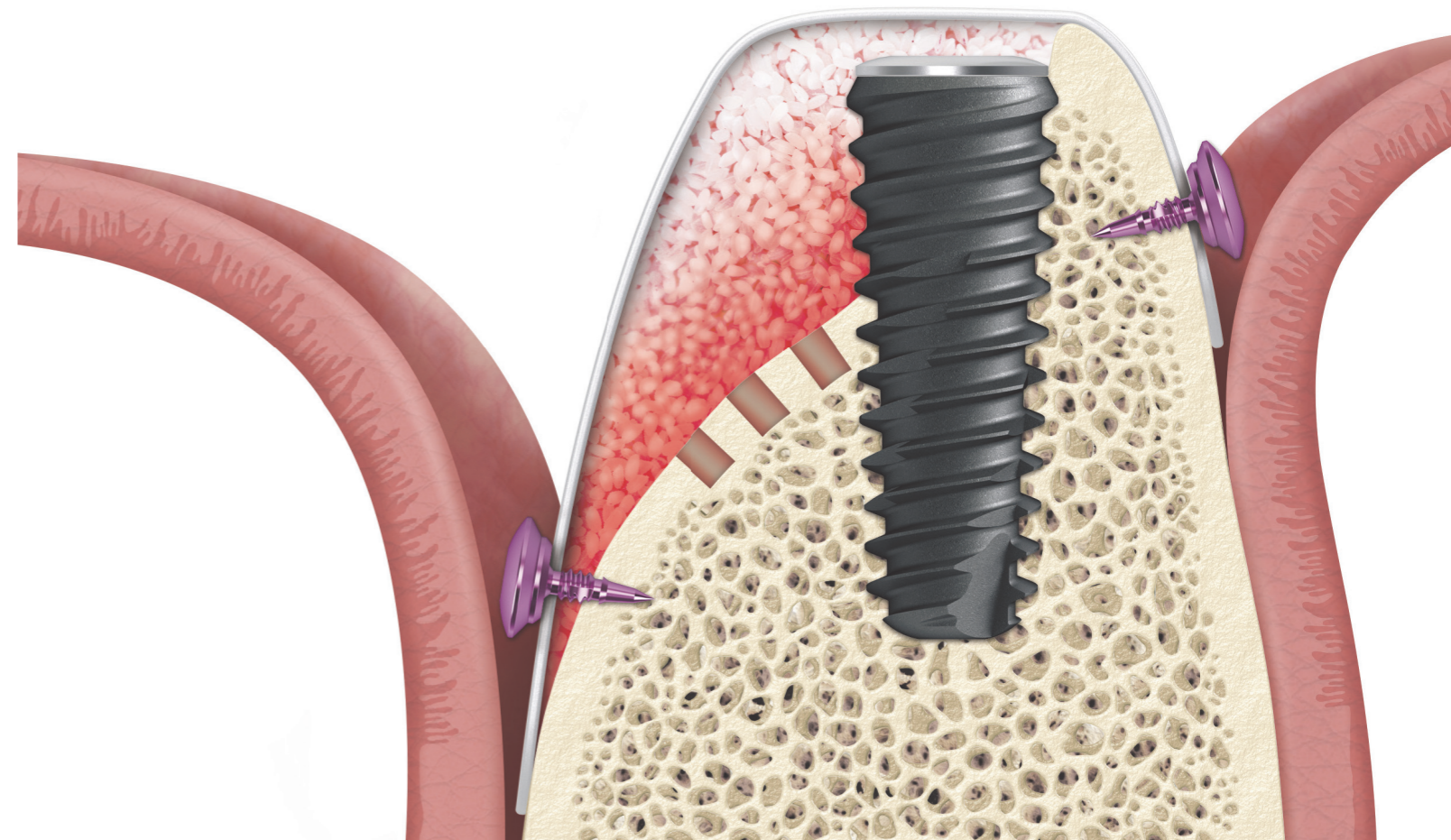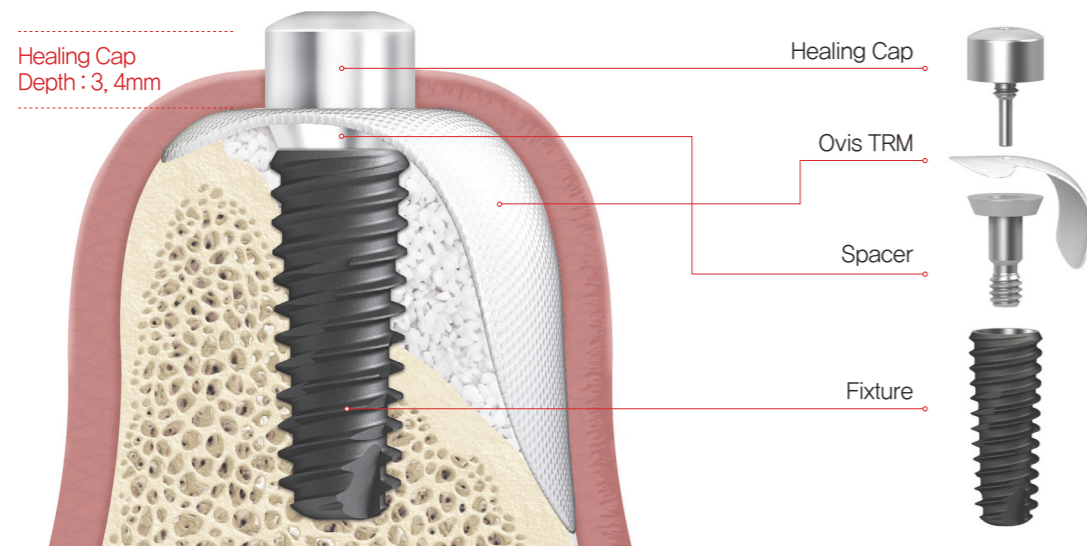

SAVE
GBR KIT
Only for GBR



Tel. +82-1899-2804 | Fax. +82-53-583-2806
99, Seongseoseo-Ro, Dalseo-Gu, Daegu, Korea



Tel. +1-323-677-4363~5 | Fax. +1-323-677-4366
6 Centerpointe Drive, Suite 600 La Palma CA 90623




Tel. +021-5111-3828 | Fax. +021-5111-3828
上海市长宁区中山西路933号2205室

©2021 DENTIS CO.,LTD. All Right Reserved.
Specifications are subject to change without notice. Trademarks are the property of DENTIS CO., LTD. Or their respective owners.

D-R-SAVEGBR-V3-202102-EN

SAVE GBR KIT

SAVE GBR KIT is composed of fixing screw such as Bone Screw and Bone Tack for GBR technique. Each of these components in the KIT are specialized for DENTIS products and provides successful GBR solution when using with Ovis Bone, SQ Fixture, OneQ Fixture.

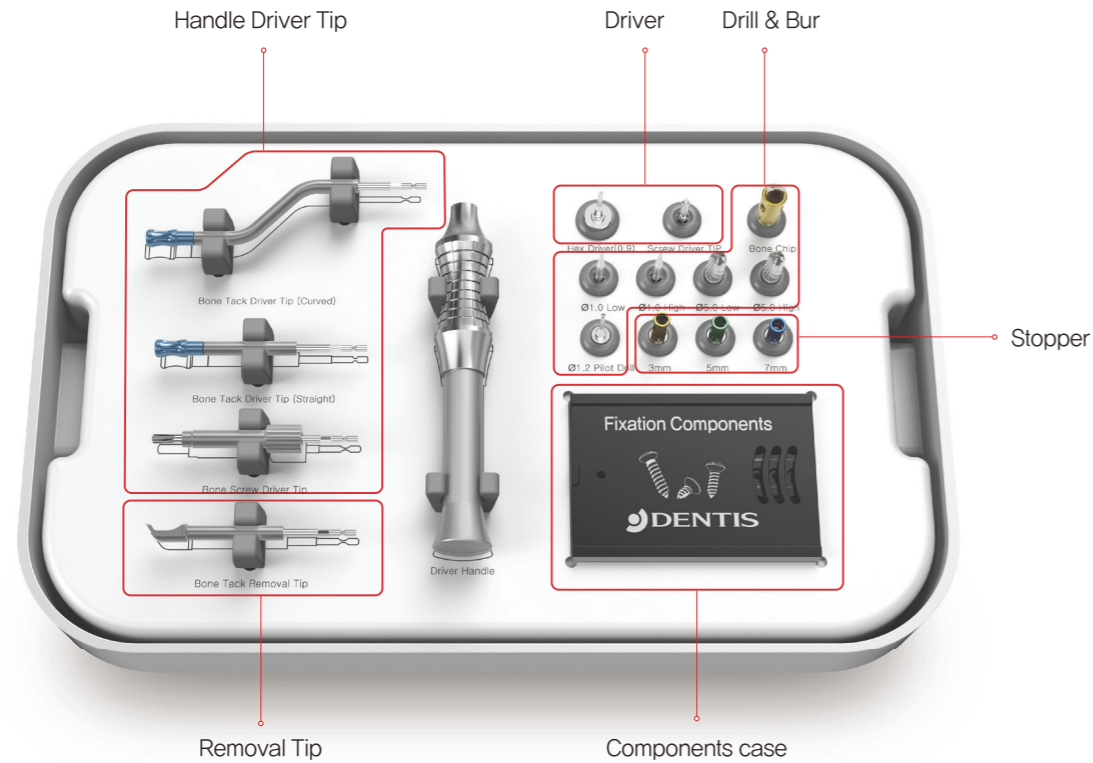

⊕ Drill & Bur



Bone Chip Drill(5.0)

- For harvesting autogenous bone. Stopper needs to be tightened to Bone Chip Drill first.
- Components : Bone Chip Drill, Stopper
- Drill Diameter : \varnothing 5.0
- Drilling depth : 4mm
- Recommended RPM : 300~600RPM

CODE : DGBC50



Cortical Perforation Bur

- For stable osseointegration by drilling cortical bone
- Drill Diameter : \varnothing 1.0
- Low speed, High speed

CODE : DGCPB10L(Low)
DGCPB10H(High)



Bone Trimming Bur

- For filing Alveolar bone and removing granulation of bone defect part.
- Drill Diameter : \varnothing 5.0
- Low speed, High speed

CODE : DGBTB50L(Low)
DGBTB50H(High)

⊕ Handle & Driver



Hex Driver(0.9)

- For tightening Spacer and Healing Cap

CODE : DRHDL09



Driver Tips for Universal Handle (Bone Tack & Bone Screw)

- Universal handle connecting with Bone Screw Driver, Tack Driver and Removal Tip.

CODE : DGUDH

⊕ Bone Screw & Tenting Screw Instruments



Pilot Drill

- For thick cortical bone when using Bone Screw and Tenting Screw.
- Drill Diameter : \varnothing 1.2mm
- Recommended RPM : 800~1500RPM

CODE : DGPD12



Stopper/ Holder

- Connected to Pilot Drill to have control for Drilling depth when implanting Bone Screw and Tenting Screw.
- Length : 7/5/3mm

CODE : DGS07/DGS05/DGS03
DGSH(holder)



Bone Screw Driver

- For holding and implanting Bone Screw
- Universal Handle

CODE : DGBSD



Bone Screw Driver Tip(For H/P)

- For holding and implanting Bone Screw
- For Handpiece


CODE : DGBSDTH

⊕ Bone Tack Instruments

Bone Tack Driver(Straight/Curved)

- For holding and implanting Bone Tack

CODE : DGBTDS(Straight)
DGBTDC(Curved)

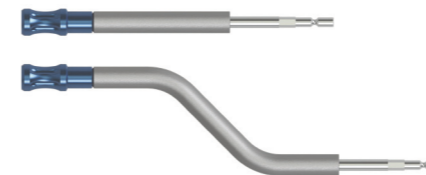

Bone Tack Removal Tip

- For removing fixed Bone Tack

CODE : DGBTRT