

ChecQ PEG Reference Guide

No. PEG Type

* Under Production

Manufacturer	Model	Size	No.	
DENTIS	s-Clean	SQ Mini	Ø3.0	11
			Ø3.5 / Ø4.0	05
		SQ Regular		11
		SQ Wide		11
		OneQ Narrow		22
		OneQ Regular		11
		OneQ Wide		11
		Cleanlant Mini	Ø3.7	11
		Cleanlant Regular	Ø4.1 / Ø4.3	11
		Cleanlant Wide	Ø4.8	11
	SQ SAVE Submerged	Ø5.5 / Ø6.0	07	
	i-Clean	TQ, OneQ, Cleanlant - Regular, Wide		04

Manufacturer	Model	Size	No.
MEGAGEN	AnyOne Regular		07
	AnyRidge		27*
	Ari ExCoN		02*
	BLUE DIAMOND Narrow		53*
	BLUE DIAMOND Regular		54*
	EZ Plus Internal		07
	MINi		50*
	ST Mini		28*
	ST Regular		06

Manufacturer	Model	Size	No.
OSSTEM	KS III Reg	7mm +	38*
		6mm	63*
	SS II		04
	SS III		04
	TS II Mini		05
	TS II Reg		61
	TS III Mini	Ø3.0	11
		Ø3.5	05
	TS III Reg	Ø4.0 - Ø5.5	61
		Ø6.0 +	04
TS IV Reg	Ø4.0 - Ø5.5	61	
	Ø6.0 +	04	

Manufacturer	Model	Size	No.
DIO	Bone Level UF (II), (III) Narrow		22
	Bone Level UF (II), (III) Regular, Wide		06
	Tissue Level UF (II)		04

Manufacturer	Model	Size	No.
STRAUMANN	BLX Implant Roxolid		38*
	Bone Level Implants Roxolid	NC	53*
		RC	54*
	Bone Level Tapered Implants Roxolid	NC	53*
		RC	54*
		SC	22
	Standard Implants Roxolid	RN, WN	04
		NNC	68*
	TLX S Roxolid	NT	95

Manufacturer	Model	Size	No.
DENTIUM	SuperLine		07
	SimpleLine		07
	SimpleLineII		04
	Implatinum		06

Manufacturer	Model	Size	No.
IBS	Magicore		95
	Magic FC		06
	Magic FC Mini		05


Manufacturer	Model	Size	No.
NEO	IS-II active Fixture (S.L.A Surface)		05

Manufacturer	Model	Size	No.
POINT	SM2 Mini		05
	SM2 Regular		06

Instruction


1. Fasten ChecQ PEG

Attach the PEG that matches the implant specifications using the ChecQ PEG Driver.

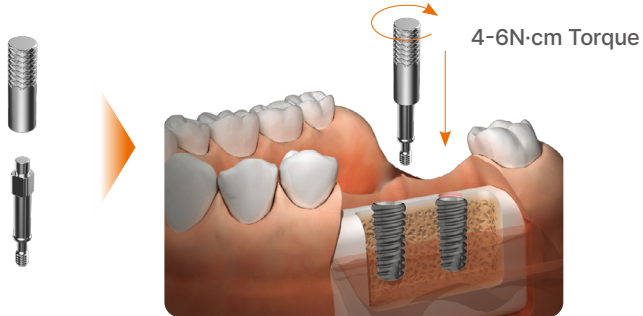

2. Measure the Implant Stability

Place the powered-on ChecQ device 2~3mm from the ChecQ PEG. When a response is detected, a beeping sound will play while displaying the measurement values.

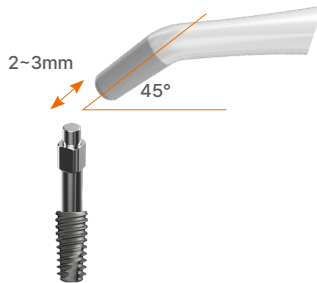

3. Remove PEG

Once the measurement is complete, use the ChecQ PEG Driver to detach the ChecQ PEG.

